teesa®

Blood pressure monitor

BPM 100

TSA8045

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DE

Owner's manual

EN

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Thank you for purchasing TEESA appliance. Please read this operation instruction carefully before use, and keep it for future use. Distributor does not take responsibility for damages caused by inappropriate handling and use of the product.

SAFETY INSTRUCTIONS

In order to reduce the risk of fire, electric shock, or any other injury, follow basic safety precautions applied when using any electrical device, including the following:

1. Read this instruction manual carefully, even if you are familiar with the appliance. Keep the manual for future reference.
2. Product intended for household use only.
3. Do not use the product for purposes different from displayed in this operating manual.
4. Clean and store this appliance in accordance to instructions listed in "Cleaning" and "Storage" section.
5. ALWAYS turn off the device:
 - if it's not operating correctly,
 - if there's an uncommon voice when using,
 - before disassembling the device,
 - before cleaning,
 - when not in use.
6. This appliance must not be used by children and by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge. Children should not play with this device. Children should not perform cleaning and servicing of the device unsupervised.
7. Keep the appliance out of the reach of children.
8. Place the appliance on a dry, flat and stable surface.
9. Do not drop the device or subject it to strong impacts.
10. Do not place the appliance in water or any liquid; nor handle/use it with wet or moist hands.
11. Use only authorized accessories.
12. Do not attempt to repair this appliance yourself. Only authorized and qualified personnel may repair this appliance. Never disassemble this appliance.
13. In case of damage, contact with an authorized service point for check-up/repair

SAFE USE

The below warning signs and icons were used to ensure safe and proper use of the device and to prevent any injuries and damages to the device.

This icon indicates prohibitions.

This icon indicates something that is compulsory.

This icon indicates the prohibition to disassemble the device.

Type BF applied part

This icon indicates that the medical device has to be protected from humidity, water and other liquid.

MR Unsafe
Do not use this
equipment in the MR
scan room

The following symbol indicates that the medical device is MR-unsafe. It is forbidden to use the device in MRI environment (MR system room).

EN

	Always follow doctor's instructions and do not perform self-judgment and self-treatment on the basis of the measurement.
	Self-diagnosis and self-treatment may be dangerous.
	The device cannot be used to diagnose illness and is not suitable for continuous monitoring during first aid and surgery.
	The device cannot be used during transportation. Device for indoor use only. Turn off the device when you experience discomfort during measurement or when the air is inflating abnormally without stop.

	This device cannot be used by children below 12 and by people who cannot express their intentions. If the device is used by people between 12 - 18, make sure they are supervised by an adult.
	Do not use the device for purposes different than measuring blood pressure.
	Do not use mobile phones when operating the device. Do not expose the device to magnetic field.
	Do not move during measurement.
	Do not disassemble and repair the main unit and the cuff. Risk of electric shock and damaging the device.

INTENDED USE

This device uses the oscillometric method of blood pressure measurement. This blood pressure monitor is intended for measuring diastolic and systolic blood pressure and pulse rate among adults at home as well as in a medical facility. To measure blood pressure, you have to attach the cuff to the upper arm (recommended left) as described in the section "Attaching the cuff" and turn on the device.

NOTES

- To prevent injuries, before taking measurement make sure that air tube is not tangled.
- Do not measure more than 3 times at a time. Minimal interval between each measurement is 5 minutes. Continuous measuring may cause bruising.
- Do not measure more than 6 times a day.
- Do not attach the cuff to the injured arm, as it may lead to further injuries.
- Do not measure blood pressure on the arm on the side of a mastectomy as it may lead to further injuries.
- Observe the measurement progress from the display.
- If pressure in the cuff exceeds 280 mmHg; immediately turn off the device by pressing the ON/OFF button.
- Do not bend the cuff or the air tube.


BEFORE FIRST USE

1. If taking medication, first consult the doctor to determine the most appropriate time for measurements. Do not change the dose of prescribed medication without consulting the doctor.
2. For people with peripheral circulation insufficiency due to diabetes, liver disease, hardening of the arteries etc. the results may be different for measurement taken on the arm and on the wrist.
3. Electrical devices such as TV set, microwave, X-ray machine, mobile device or other devices with strong electromagnetic field may interfere with the blood pressure monitor and display incorrect results. Use this blood pressure monitor away from such devices or make sure they are turned off.
4. Make sure to wash your hands before taking measurement.
5. Do not use the device with other medical electrical equipment simultaneously.
6. If disturbing results are obtained, consult your doctor or refer to "Troubleshooting" section.
7. Measurement result may be higher in doctor's office.

PRODUCT DESCRIPTION


BLOOD PRESSURE MONITOR

1. Cuff
2. Air tube
3. Air tube plug
4. MEM button
5. SET button
6. ON/OFF button
7. LCD display
8. Cuff socket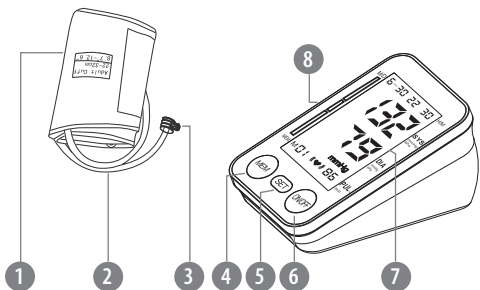


DISPLAY

1. Month/day
2. Blood pressure classification indicator
3. Low battery level icon
4. Inflation indicator/ Deflation indicator
5. Memory
6. Irregular heartbeat detection icon
7. Pulse icon
8. Pulse
9. Pressure unit
10. Diastolic blood pressure
11. Systolic blood pressure
12. Hour/minute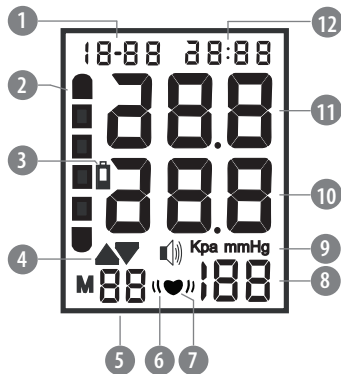

BLOOD PRESSURE CLASSIFICATION ACC. TO WHO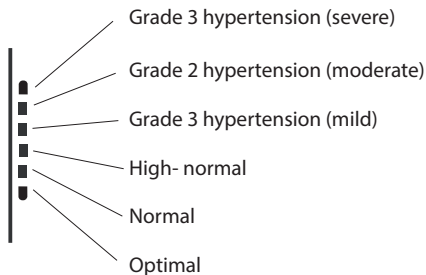

BATTERY INSTALLATION


1. Remove the battery compartment cover.
2. Insert the batteries with correct polarization (+/-).
3. Close the battery compartment cover.

Notes:

- Use only specified batteries.
- Do not mix batteries of different type.
- Do not insert the batteries with signs of leakage as it may damage the device.
- Do not short-circuit the batteries.
- Replace the batteries with the new ones when the low battery level icon flashes.
- Replace all batteries at the same time.
- Battery lifespan depends on the ambient temperature and it may be shorter in environments with lower temperature.
- Store the device without the batteries inside if it is not going to be used for more than 3 months.

SETTING DATE AND TIME


1. Press and hold the SET button until year flashes on the display.
2. Keep pressing the MEM button to set the year; press the SET button to save and set the month.
3. Keep pressing the MEM button again to set the month. Follow the same steps to set day, hour and minute.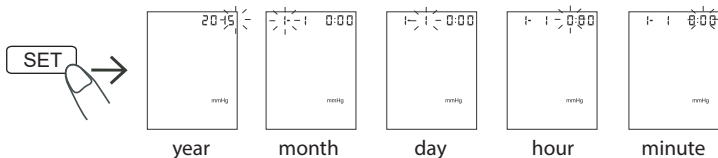

USER SELECTION

When the device is turned off, press the SET button to select a user.


*** To display stored results for a certain user, first select a user, then press the MEM button to display stored results.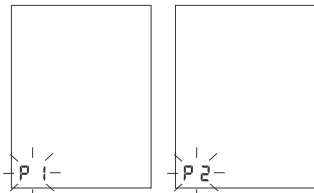

SETTING BLOOD PRESSURE UNIT


The device has two pressure units for choosing: mmHg and kPa (mmHg by default).

Press and hold the ON/OFF button for about 10 seconds until pressure unit interface appears on the display. Press the MEM button to set desired unit.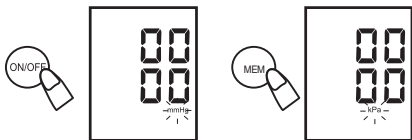


ATTACHING THE CUFF

1. Connect the air tube plug to the cuff socket.
2. Attach and wrap the cuff around the upper arm, directly on the skin with its lower part approximately 2-3 cm above the bend of the elbow; the air tube should point towards the hand as shown on the drawing below.
 - Do not attach the cuff on clothes as it may lead to incorrect readings.
 - Arm constriction caused by rolling up shirt sleeve may affect the accuracy of the measurement.
3. Fasten the cuff closely but not too tight so that you feel comfortable.
4. Support your arm on the table, with the palm facing upwards.
5. Make sure the cuff is at the same height as your heart.
6. Make sure the air tube is not tangled.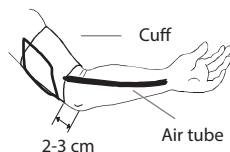


CORRECT POSTURE AND BEHAVIOR DURING MEASUREMENT

1. To achieve correct measurement result, adopt appropriate body posture, that is:
 - sit comfortably, preferably on a chair, lean back,
 - straighten your back
 - put your feet on the floor,
 - do not cross your legs,
 - support your arm, for example on a table,
 - make sure the cuff is at the same height as your heart.
2. Relax.
3. Do not talk or move.
4. Take measurement at the same time each day.
5. Do not take measurement directly after physical activity or bath. Wait at least 20-30 minutes before taking measurement.
6. Avoid the below situations to prevent incorrect results:
 - taking measurement within an hour from eating, drinking alcohol, coffee, tea,
 - taking measurement within an hour from physical activity and bath,
 - talking,
 - being nervous or anxious,
 - leaning forward,
 - moving,
 - rapid temperature changes in a room during measurement,
 - taking measurement during transport,
 - continuous measuring.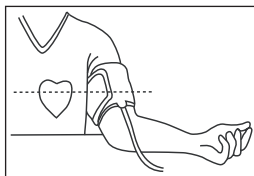


USING THE DEVICE

1. Attach the cuff to the arm as described in the "Attaching the cuff" section.
2. Press the ON/OFF button; the device is ready for taking measurement when 00 appears on the display.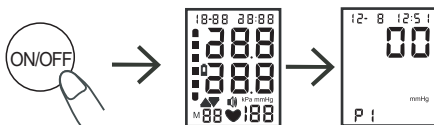

3. The cuff will be inflated automatically; pulse icon flashes.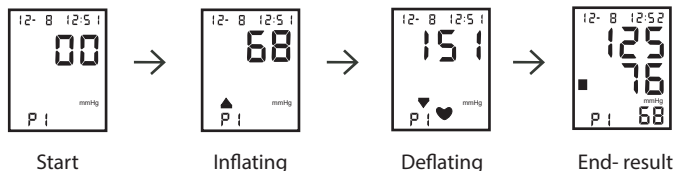


4. When the measurement is finished, systolic and diastolic blood pressure, pulse rate and classification of achieved result is displayed. Each result is stored in the device's memory together with time and date.

READING STORED RESULTS

Press the MEM button to enter memory. Keep pressing the MEM button to switch to the next result; keep pressing the SET button to switch to the previous result.

DELETING ALL RESULTS

Press and hold the MEM button for about 5 seconds to delete all results.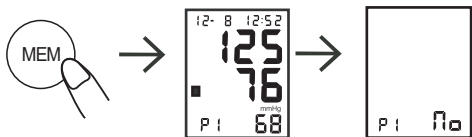

5. Remove the cuff and turn off the device by pressing the ON/OFF button. The device will automatically turn off after 1 minute.
6. When you disconnect the air tube from the device, pull the plug, not the tube.
7. Store the device in the storage case included in the set.

CLEANING AND STORAGE

- Clean the device and the cuff with a soft, dry cloth.
- Do not use any chemical agents to clean this device.
- Do not fold the cuff too tightly.
- If the device is not in use, store it in the storage case.
- Store the device in a clean and dry place.

To ensure proper operation and performance, it is recommended to service and calibrate the device every 2 years.

ENVIRONMENTAL CONDITIONS

The device may not meet its peak performance if it is stored or used in conditions other than those described below:

Operating conditions: +5°C~+40°C; 15%~93%; 70 kPa~106 kPa

Storage conditions: 5°C~+40°C; 15%~93%; 70 kPa~106 kPa

CLASSIFICATION

1. Type of protection against electric shock: **INTERNALLY POWERED EQUIPMENT.**
2. Degree of protection against electric shock: **TYPE BF APPLIED PART.**
3. Operation mode: **CONTINUOUS OPERATION.**
4. This device is not AP/APG equipment and is not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.

TROUBLESHOOTING

ERROR	POSSIBLE CAUSE	POSSIBLE SOLUTION
After pressing the ON/OFF button, the display does not turn on or low battery icon flashes	Batteries are not installed	Install the batteries in the battery compartment
	The batteries are discharged	Replace batteries with the new ones
	The batteries are installed incorrectly	Install the batteries with correct polarization
E1: Cuff pressure does not rise	The cuff is leaky	Replace the cuff with a new one
E3: Pressure in the cuff is too high	Pressure exceeded 299 mmHg	Contact authorized service point for check-up/repair
E2E4: Movement during measurement	Movement during measurement	Do not move during measurement
Low battery icon is displayed	Battery level is low	Replace the batteries with the new ones and take another measurement
Systolic or diastolic blood pressure too high	The cuff has been attached below the heart level	Follow the instructions described in the "Attaching the cuff" and "Correct posture and behavior during measurement" sections and take another measurement
	The cuff has been attached incorrectly	
	Movement or talking during measurement	
Systolic or diastolic blood pressure too low	The cuff has been attached above the heart level	
	Movement or talking during measurement	

SPECIFICATION

Oscillometric measurement

One-touch operation

Blood pressure indicator according to WHO

Large backlit digital LCD display

Irregular heartbeat detection

Blood pressure measurement range: 30 – 280 mmHg

Pulse measurement range: 40 – 199 beats/min.

Blood pressure measurement accuracy: +/- 3 mmHg

Pulse measurement accuracy: +/- 5%

Cuff circumference: 22 - 32 cm

Date and hour function

Stores 2x 90 measurements with date and time for two users

Automatic power off after 1 minute

Low battery indicator

Classification: BF Type

Protection class: IP20

Operating temperature: 5°C – 40°C

Operating humidity: 15 - 93%

Storage temperature: -20°C – 55°C

Storage humidity: 0 - 93%

Power supply: 4x AA 1,5 V battery (approx. 200 measurements)

In set: arm cuff, storage case, user's manual

Dimensions: 95,9 x 78 x 150 mm


Weight: 445 g (without batteries)

The device is consistent with the requirements of the Medical Device Directive 93/42/EEC.

English

Correct Disposal of This Product
(Waste Electrical & Electronic Equipment)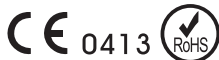

(Applicable in the European Union and other European countries with separate collection systems) This marking shown on the product or its literature, indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

Made in China for LECHPOL ELECTRONICS Sp. z o.o. Sp.k., ul. Garwolińska 1, 08-400 Miętno.

Shenzhen Pango Electronic Co., Ltd.

Main Site: No.25 1st Industry Zone, Fenghuang Road, Xikeng Village, Henggang Town, Longgang District, Shenzhen, Guangdong China

Additional site1: 2-4 Floor, No.5 Shanzhuang Rd., Xikeng Village, Henggang Town, Longgang District, Shenzhen City, Guangdong Province, China

Authorised representative: Lotus Global Co., Ltd.

Address: 1 Four Seasons Terrace West Drayton, Middlesex London, UB7 9GG United Kingdom

Tel: 0044-20-75868010, 0044-20-70961611

Fax: 0044-20-79006187

EC	REP
----	-----

Appendix 1 Guidance and Manufacturer Declaration Tables

Guidance and manufacturer's declaration- electromagnetic emission		
The Model TSA8045 is intended for use in the electromagnetic environment specified below. The customer or the user of the Model TSA8045 should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Model TSA8045 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	A	The Model TSA8045 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complied	

Guidance and manufacturer's declaration- electromagnetic immunity			
The Model TSA8045 is intended for use in the electromagnetic environment specified below. The customer or the user of the Model TSA8045 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance

<p>Electrostatic discharge (ESD) IEC 61000-4-2</p>	<p>± 8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air</p>	<p>± 8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air</p>	<p>Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.</p>
<p>Electrostatic transient/burst IEC 61000-4-4</p>	<p>±2 kV for power supply lines 100 kHz repetition frequency ±1 kV for input/output lines</p>	<p>±2 kV for power supply lines 100 kHz repetition frequency ±1 kV for input/output lines</p>	<p>Mains power quality should be that of a typical commercial or hospital environment.</p>
<p>Surge IEC 61000-4-5</p>	<p>±0.5 kV, ±1 kV differential mode line-line</p>	<p>±0.5 kV, ±1 kV differential mode line-line</p>	<p>Mains power quality should be that of a typical commercial or hospital environment.</p>
<p>Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11</p>	<p>0% UT (100% dip in UT) for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% UT (100% dip in UT) for 1 cycle at 0° 70% UT (30% dip in UT) for 25/30 cycles at 0° 0% UT (100% dip in UT) for 250/300 cycle at 0°</p>	<p>0% UT (100% dip in UT) for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% UT (100% dip in UT) for 1 cycle at 0° 70% UT (30% dip in UT) for 25/30 cycles at 0° 0% UT (100% dip in UT) for 250/300 cycle at 0°</p>	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of the Model TSA8045 product name requires continued operation during power mains interruptions, it is recommended that the Model TSA8045 be powered from an uninterruptible power supply or a battery.</p>

Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m,50/60H	30 A/m,50/60Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: UT is the a. c. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration-electromagnetic immunity

The Model TSA8045 is intended for use in the electromagnetic environment specified below. The customer or the user of the Model TSA8045 should assure that it is used in such an electromagnetic environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz 6 Vrms 150 kHz to 80 MHz outside ISM bands	6 V	Portable and mobile RF communications equipment should be used no closer to any part of the Models TSA8045, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$

$$d = \left[\frac{12}{V_2} \right] \sqrt{P}$$

$$d = \left[\frac{12}{E_1} \right] \sqrt{P} \text{ 800MHz to 800MHz}$$

$$d = \left[\frac{23}{E_1} \right] \sqrt{P} \text{ 800MHz to 2.7GHz}$$

Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres(m).

Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance

level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a The ISM (industrial, scientific and medical) bands between 0,15 MHz and 80 MHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.

b The compliance levels in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range 80 MHz to 2,7 GHz are intended to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas. For this reason, an additional factor of 10/3 has been incorporated into the formulae used in calculating the recommended separation distance for transmitters in these frequency ranges.

c Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the TSA8045 is used exceeds the applicable RF compliance level above, the TSA8045 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the TSA8045.

d Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10V/m.

Recommended separation distances between portable and mobile RF communications equipment and the Model TSA8045

The Model TSA8045 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Model TSA8045 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Model TSA8045 as recommended below, according to the maximum output power of the communications equipment.

Separation distance according to frequency of transmitter m

Rated maximum output of transmitter	150 kHz to 80 MHz outside ISM bands	150 kHz to 80 MHz in ISM bands	80 MHz to 800 MHz	800 MHz to 2,7 GHz
W	$d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$	$d = \left[\frac{12}{V_2} \right] \sqrt{P}$	$d = \left[\frac{12}{E_1} \right] \sqrt{P}$	$d = \left[\frac{23}{E_1} \right] \sqrt{P}$
0.01	0.12	0.12	0.12	0.23
0.1	0.38	0.38	0.38	0.73
1	1.2	1.2	1.2	2.3
10	3.8	3.8	3.8	7.27
100	12	12	12	23

For transmitters rated at a maximum output power not listed above the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 The ISM (industrial, scientific and medical) bands between 0,15 MHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0, 15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz; 3,5 MHz to 4,0 MHz; 5,3 MHz to 5,4 MHz; 7 MHz to 7,3 MHz; 10,1 MHz to 10,15 MHz; 14 MHz to 14,2 MHz; 18,07 MHz to 18,17 MHz; 21,0 MHz to 21,4 MHz; 24,89 MHz to 24,99 MHz; 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.

NOTE 3 An additional factor of 10/3 has been incorporated into the formulae used in calculating the recommended separation distance for transmitters in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range 80 MHz to 2,7 GHz to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas.

NOTE 4 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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