ALKOHOLTESTER KEMOT MIE2125

1. BEDIENUNG

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BREATHALYZER KEMOT MIE2125

1. OPERATION

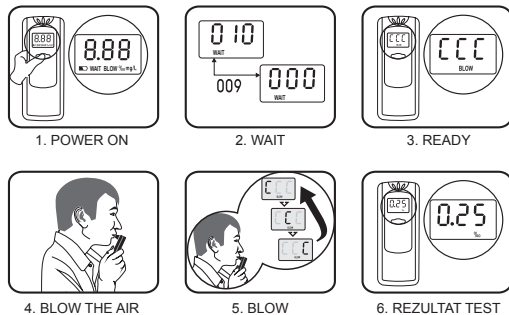
- **Power-on:** Press the switch for 2 second.
- **Ready:** After countdown to zero, the LCD display „CCC“.
- **Blow:** Blow for 3 seconds into the holes at the top of the device (keep a distance 5- 10 mm between your mouth and the product).
- **Test Result:** The LCD displays the test results and powers off automatically afterwards.
- **Retest:** Wait for 30 seconds.

2. BATTERIES

Use 2 pcs AAA alkaline batteries. Insert the new batteries when the LCD displays .

CAUTION!

The environment will affect the test accuracy, and the alcohol tester gives BAC for reference only. Do not drink and drive! The producer does not take any legal responsibility. For accuracy, test after 15 minutes after last drink.

1. POWER ON

2. WAIT

3. READY

4. BLOW THE AIR

5. BLOW

6. REZULTAT TEST

English
Correct Disposal of This Product
(Waste Electrical & Electronic Equipment)

(Applicable in the European Union and other European countries with separate collection systems) This marking shown on the product or its literature, indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

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